

SUMMARY

- To write a message Press: ON/C and then your message.
- To repeat the message Press: REPLAY.
- To delete the last letter Press: BACK-SPACE.
- To clear the display Press: ON/C.
- To summon attention Press: BUZZ.
- To recall a memory Press: MEM then L (for example) ... first, store a message!
- To turn Lightwriter off Press: OFF.

**USE ALL DAY, AND RE-CHARGE OVERNIGHT: USE THE CHARGER SUPPLIED.
IF THE BATTERY IS LOW, USE THE LIGHTWRITER WITH THE CHARGER PLUGGED IN**

TRY EACH OF THESE WITH THE MACHINE IN FRONT OF YOU!

KEYBOARD

- Shift-once Press: SHIFT once to shift the following character only.
EXAMPLES: **SHIFT FULLSTOP TO GET COMMA, SHIFT 5 TO GET %**
- Shift-lock Press: SHIFT twice to shift all characters; press a 3rd time to unlock.
NOTE: **IF THE SETUP IS "lower-case", SHIFT ALSO PRODUCES UPPER-CASE LETTERS**

DISPLAYS

Case You can type in UPPER or lower case; see SETUPS.

REPLAY

This key will repeat and speak the message that you have typed; the speed of Replay is controlled by the setup Speed .
You can type indefinitely, but you can only recall the first 255 characters of a message.
To speak as fluently as possible, the setup is Speed 4.

CE STATEMENT:

An Indirect-field static-discharge (eg lightning) may cause the device to 'lock up' (if it is in use at the time).
In this circumstance, the user should press OFF then ON.

AIRCRAFT

If you are advised by a stewardess not to use other electrical equipment such as portable computers, video recorders, or Walkman radios during parts of the flight such as take-off or landing, you should also refrain from using the Lightwriter during these periods.

Carry the equipment as hand-luggage and not in a suitcase or bag; show airport-security staff how it works; this may make the security-check faster. The equipment will not be affected by the security X-ray machines.

CHARGER

If you plan to travel, the existing charger will work (with a suitable adaptor), provided the supply is 220 - 250 volts (Europe, most of Australia, New Zealand, etc).

For countries like USA/Canada or Japan where the voltage is lower, you can't use the original charger; you can buy a charger for USA/Canada from us, or from our US distributor: ZYGO Industries Inc, Oregon, (503) 684 6006. If in doubt, check the voltage in the country you are going to.

The charger output must be 9-10 VDC, minimum rating of 800mA, output-plug: 2.1 mm x 5.5 mm x 12 mm, positive-tip. A car charging-lead is available; this charges the equipment from a car cigarette-lighter socket.



ACCESSORIES

KEYGUARD

If the green keyguard spacers do not provide a deep enough keyguard, we supply an alternative keyguard with deep recesses. This can be fitted by the user.

KEYBOARD GLOVE

A clear plastic sheet which fits over the keyboard.

RAINCOAT

Same but covers entire machine, and provides a reasonable amount of weather-protection. Specify serial-number.

CARRYING

Neck strap:	Clips onto the keyboard's carry-handle; supplied with the machine.
Carry bag:	Soft, padded material; supplied with shoulder-strap and waist-strap.
Wrist strap:	Carry the Lightwriter with a strap wrapped round the wrist.
Protective case:	Executive-style suitcase.
Mounting:	Our Mobilia system provides wheelchair-fixing.

STERILISATION

If it is required to sterilise the equipment, the outer surfaces may be wiped with a cold-sterilising agent.

The equipment is splashproof but not waterproof; do not allow sterilising agent or cleaning fluid to get inside.

The recommended technique is to hold equipment on end with the charger socket lowest and to clean the equipment in this position, reducing the risk of fluid entering the equipment.

Do not use strong sterilising agents such as surgical spirit.

Do not immerse in sterilising fluid.

Do not expose to elevated temperatures.


Do not autoclave.

If a potential user is suffering from a virulent condition from which the equipment cannot be adequately sterilised using the above techniques, take care to avoid the risk of infecting future users or other people handling the equipment; you may wish to use an alphabet or picture board, which can be disposed of after use.

TROUBLE-SHOOTING

WON'T CHARGE?

- Is the mains-socket OK? And is it turned on?
Is the charger-light lit? If not, the charger may be dead.
Try wiggling the wire It may have broken inside the insulation, if it has been pulled violently.
Are you using our charger? The one we supplied has a label like this (except USA) -----

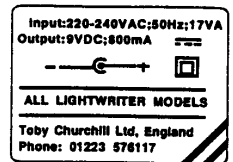

RESET

- 'Reset' on SL5 is Off/On.
If this does not cure the problem and the Lightwriter is accepting charge, it needs to be returned.

RETURNING THE EQUIPMENT

If you need to return the equipment to supplier, here is what to do:
Pack the machine and its charger with plenty of protection, preferably in the original box.

*Ensure the ON button cannot be pressed in transit.
Tape up the parcel securely.*

Return all the equipment (including charger) to your supplier.

(UK only: Toby Churchill Ltd, Unit 18, Barnwell Business Park, Barnwell Drive, Cambridge, CB5 8UY).
Address it clearly; and put your (sender's) name and address on the back. Enclose a note to say who it is from, and why.
Ring, or send a letter separately, quoting:

*Your name, address and phone-number.
The serial number (on the back).
The invoice number (if known).
The date of despatch.
The faults (or reason for return).
The courier.*

SERVICING

The Lightwriter is extremely reliable if treated properly.
By far the most frequent cause of damage is mechanical abuse (dropping it, coffee-spillage, etc).
The equipment does not need regular maintenance, and is designed to give many years trouble-free use.
We undertake to carry a full range of spare parts for this model for at least 5 years, or to supply a newer model.

INSURANCE

You may wish to insure the equipment; many companies offer policies to cover this type of equipment against Fire, Theft and
Accidental Damage. Most companies will only consider selling a complete House-contents Policy.
Consider also our 5-year Guarantee (UK only).

GUARANTEE

The Lightwriter is guaranteed for 12 months. It excludes:

*Mechanical damage, eg dropped.
Abuse.
Unauthorised removal of the back of the Lightwriter.*

We offer a 5-year Guarantee (UK only).

Manufactured by:
Toby Churchill Ltd, Cambridge
England. 01223 576117
Supplied by:

Software: V1.5
(MEM MEM MEM V to show the version)



dti
Languages for
Export Award
1996

